Gebrauchsanweisung: ABC Textil-Brustprothesen
Instructions for use: ABC Textile Breast Prostheses

Gebrauchsanweisung • Instructions for use • Gebruiksaanwijzing • Instruzioni • Instrucciones de uso • Anvendelsesinstruktion • Käyttöohjeet • Användarinstruktion • Bruksanvisning • Návod k použití

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 trocken lagern

Entsorgung

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Thank you for choosing ABC Breast Care. ABC products are developed and manufactured with great care and knowledge of the needs of breast-operated women.

This instruction manual applies to ABC textile breast prostheses.

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Please note that not all ABC products are available in all countries.

Scope of delivery:

- ABC textile breast prosthesis
- Instruction manual

Intended Use/Indication

- ABC breast prostheses are for external use only.
- ABC textile breast prostheses are intended to replace removed breast tissue after mastectomy and/or lumpectomy.
- ABC textile breast prostheses are used for post-operative initial care in the first weeks after surgery, during radiation treatment and at night for sleeping.
- ABC textile breast prostheses are worn and positioned on the body in specially made bras and/or postoperative compression bandages or compression bras with sewn-in textile pockets.
- The appropriate breast prosthesis and specialty bra or compression bandage are selected and fitted by trained professionals according to the patient's anatomical features and needs.
- ABC breast prostheses and partial breast prostheses are intended for daily use and for wearing over longer periods of time.

Risks & Contraindications

- There are no known risks or contraindications to be expected when wearing ABC breast prostheses and partial breast prostheses under normal conditions and in accordance with the care instructions.
- No safety issues or performance impairments are expected during normal use. No restrictions on use are expected.
- ABC breast prostheses are not to be used on or against on open wounds or sensitive skin.
- If you experience redness or rashes, please discontinue using the product and contact your doctor or medical device retailer.

Material

- ABC breast prostheses are carefully controlled.
- ABC textile prostheses are filled with different materials.
- The polyester and/or cotton fabric cover is a soft, clean and easily washed outer lining.
- **909 ABC First Form Fiberfill & 910 ABC Puff Pad**

ABC 909 and 910 'puffs' are made of polyester fiberfill encased in a polyfoam pad and polyester/cotton cover. The polyester fiberfill allows for individual adjustment of size and shape. The polyfoam pad gives a breast shape to the 'puff'.

Polyester Fiberfill: 100% Polyester

Polyfoam Pad: 100% Polyfoam

Cover: 100% Cotton back / 100% Polyester front

- **916 ABC First Form**

The memory foam prostheses are made of a memory foam pad encased in a nylon and cotton envelope. The memory foam pad is breast shaped.

Memory Foam: 100% Memory Foam

Cover: Front: 84% Nylon, 16% Spandex, Back (against chest): 92% Cotton, 8% Spandex

- **932 ABC Active Form**

The 932 'puff's are made of microbead fill encased in a fabric envelope. The microbeads allow for a comfortable fit.

Microbeads: 100% Thermoplastic, Cover: 100% Polyester

Instructions for use

- The textile breast prosthesis is resistant to sweat, moisture, water (both salt and chlorine water).
- Avoid stretching, twisting, friction or excessive use of force when handling your breast prosthesis.
- Avoid direct contact with hot surfaces or hot water.

Cleaning Instructions

Foam Form

- Spot wash by hand
- Do not use soap
- Rinse thoroughly, carefully squeeze out excess water and line dry.

(Note: please allow enough time for foam to dry.)

Cover & Fiberfill

 Hand wash warm  Do not bleach Line dry  Do not iron Wash before wear

Warranty

- When worn under normal conditions and in compliance with the instructions for use, we guarantee a product service life of two (2) years.
- Should your breast prosthesis become defective, please contact your medical device retailer. In the event of a production or material defect, ABC will replace the breast prosthesis should the defect happen within two (2) years from the date of purchase. The 2-year warranty does not apply to 932 ABC Active Form.

Incident Reporting

Only serious incidents relating to the product, which lead to a serious deterioration of the state of health or death, must be reported to the manufacturer or competent authority of the EU Member State. Serious incidents are described in Article 2(65) of Regulation (EU) 2017/745 (MDR).

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ABC Breast Care GmbH