ABC Breast Care

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INSTRUCTIONS FOR USE - ENGLISH

Thank you for choosing ABC Breast Care. ABC products are developed and manufactured with great care and knowledge of the needs of women after breast surgery.

This instruction manual applies to:

- ABC Compression Bandage, REF 519
- ABC Recovery Bra, REF 520
- ABC Compression Belt, REF 919
- ABC Compression Bandage Restore, Art./REF 619

Please note that not all ABC products are available in all countries.

Scope of delivery:

- ABC Compression Bandage or Compression Bra or ABC Compression Belt
- Instructions for use manual

Intended Use/Indication

- ABC Compression Bandages, Compression Bras and Compression Belt are elastic or partially elastic, textile thoracic bandages for external use only.
- The use of ABC Compression Bandages, Compression Bras and Compression Belt is usually part of medically guided therapeutic treatment over different periods of time.
- ABC Compression devices are intended for use by a single patient over extended periods of time.
- The appropriate compression fitting is selected and adapted by trained specialists according to the anatomical conditions and needs of the wearer.
- ABC Compression Devices are designed for adolescents and adults between the age of 12 to 99+ years

ABC Compression Bandages & Compression Bras

- are intended to support the breast and relieve the surgical area after breast surgery (tumour surgery and/or plastic breast surgery).
- ABC Compression Bandages and Bras are also intended as medical aids to improve wound healing by scar compression to prevent post-operative swelling and haematomas and/or in the treatment of hypertrophic scars or keloids.
- The even compression and pressure distribution relieves the scar area and lymphatic drainage pathways. It helps to shape the breast tissue and maintain a symmetrical breast contour.
- ABC Compression Bandages and Bras are designed with wide, adjustable shoulder straps and further adjustment options, usually as an anterior closure system. They are constructed with a pocket to hold the prosthesis in place and facilitate wearing a prosthesis for an extended time.

ABC Compression Belt

- ABC Compression Belts are anatomically shaped post-op belts. The anatomical shape and elastic material support the compression effect.
- They are intended to be used in combination with a compression bandage after breast reconstruction with breast-implants to hold the implant in place and prevent its upward migration.

Risks & Contraindications

- There are no known risks or contraindications to be expected when wearing under normal conditions and in accordance with the care instructions.
- No safety issues or performance impairments are expected during normal use. No restrictions on use are expected.
- ABC compression bandages and compression belts are not to be used on open wounds or sensitive skin. Adequate wound care must be ensured.
- ABC's Compression Bandages, Compression Bras and Belt are for external use only and are not to be used on or against open wounds or sensitive skin.
- If you experience redness or rashes, please discontinue using the product and contact your doctor or medical device retailer.

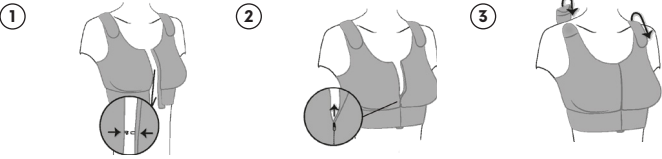
Material

- Each ABC Compression device is carefully checked for quality.
- ABC Compression Bandages, Bras and Belts are made of a combination of nylon, spandex (also denoted/known as elastane or lycra), polyester and/or cotton. The amount of each varies according to style.
- All materials have been tested for skin compatibility.

Instructions for use:

ABC Compression Bandage, Style 519 and ABC Recovery Bra, Style 520, ABC Compression Bandage Restore, Style 619


1. Put on your compression bandage and first close the hooks located under the zip.
2. Close the zipper.
3. Adjust the correct height and fit of your compression bandage using the Closure possibility on the straps.

Give your body some time to get used to your compression bandage. Wearing it on your skin may be unfamiliar at first.

ABC Compression Belt (Style 919) usually only in combination with ABC Compression Bandage (Style 519) or ABC Compression Bandage Restore (Stlye 619). Your attending physician/nurse will determine the use of the ABC Compression Belt, or the amount of compression over the chest.

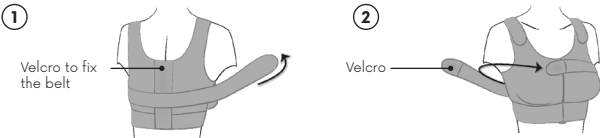1. Before putting the belt on, please make sure that the Velcro at the end of the belt is in your right hand (the fleece in your left hand). Attach the Velcro area in the middle of the belt to the back of the compression bandage.
2. Guide the belt under the armpits to the front and fasten it from the right (Velcro) to the left (fleece).

- Please note: It is not possible to securely fasten the ABC compression belt to a regular sports bra.
- Be careful with sharp, pointed objects. Brooches, needles, fingernails, scissors, pet claws, and other sharp objects can cause permanent damage to your compression device.
- Be aware that discoloration or staining, e.g. due to dark textiles or radiation markings, will not affect the functionality or quality of the ABC Compression Bandage, Bra or ABC Compression.
- If the function of your ABC compression device is impaired, please contact your medical retailer.
- A regular check up by qualified staff is recommended.

Cleaning Instructions:

- Thorough and soft cleaning is highly advised as the use of skin care products, sweating or other environmental impacts may affect product condition and performance.
- Avoid stretching, twisting, friction or excessive use of force when handling your compression device.

Close Velcro fasteners (strap and belt ends) and zipper before cleaning

 Machine wash max.40°C	 Do not bleach	 Line dry	 Do not iron	 Wash before wear
--	---	--	---	--

Incident Reporting

Only serious incidents relating to the product, which lead to a serious deterioration of the state of health or death, must be reported to the manufacturer or competent authority of the EU Member State. Serious incidents are described in Article 2(65) of Regulation (EU) 2017/745 (MDR).

Warranty

The statutory warranty obligation applies. If treated in accordance with the care instructions, the compression textiles retain their medical effect for the full 6-month period of use.

Additional information

-  Consult Instructions for use (for EU only)
-  Medical Device
-  Single patient – multiple use
-  Non sterile

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ABC Breast Care GmbH